

Installasjonskvalifisering og oppfølging av maskinelt utstyr sett fra leverandørens perspektiv


Dekontamineringsdagene 2025

Are Blyverket, 14.05.2025




We make **life-saving
technology** accessible
for more people

Livsløp

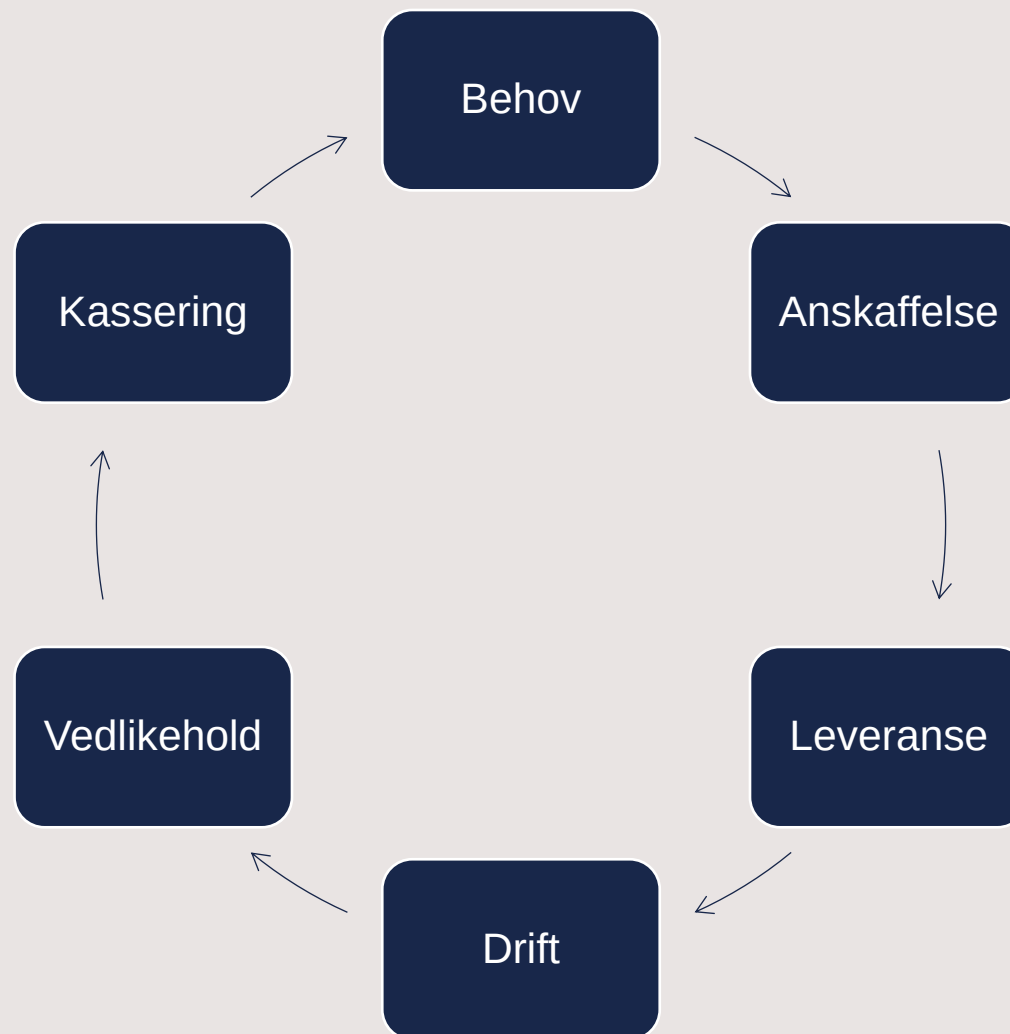



Behov

- Krav til maskinelt utstyr og leveranse.
- Analyse av behov for kapasitet.
- God oversikt over alle muligheter for kvalitetssikring og oppfølging?

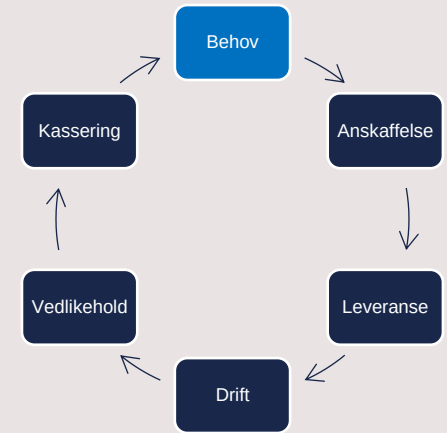


Anskaffelse


- Krav fra behovsanalysen ivaretas.
- Krav som støtter kvalitetssikring og oppfølging?

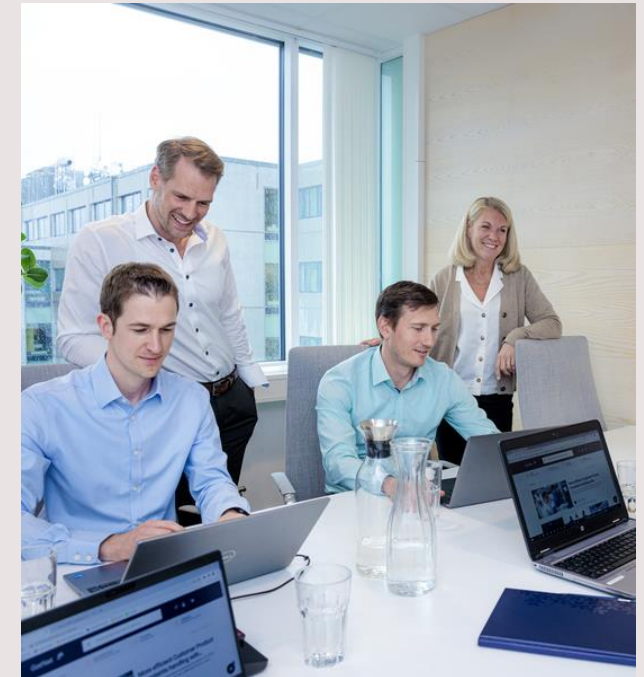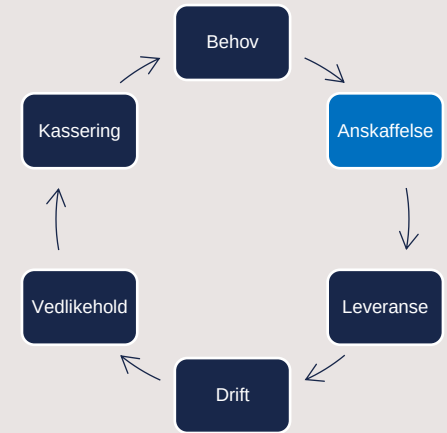

Leveranse

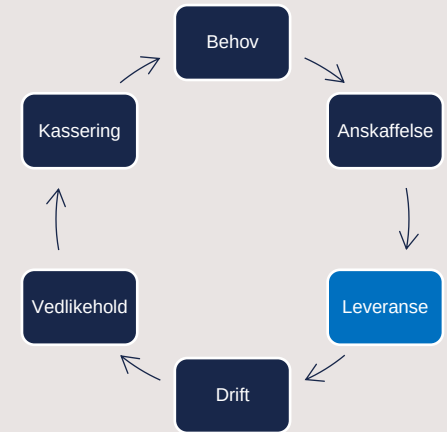


- Montering, igangkjøring, installasjonskvalifisering (IQ)
- OQ, PQ
- Opplæring
 - Brukere
 - E-læring
 - Teknikere

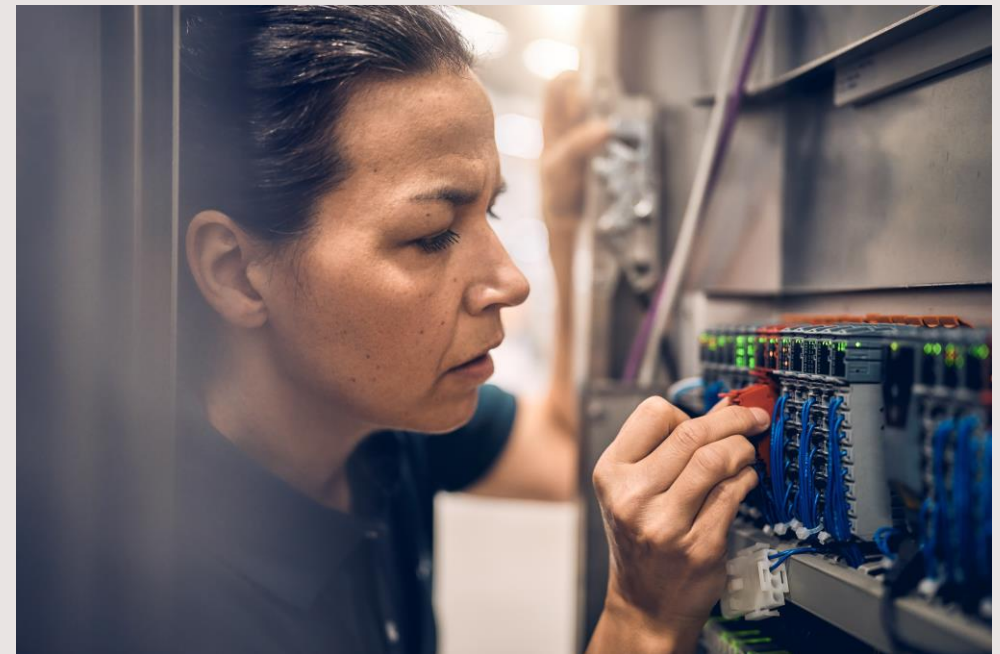

Drift -2

Digitale hjelpemidler/støtte – Fleetview og T-DOC

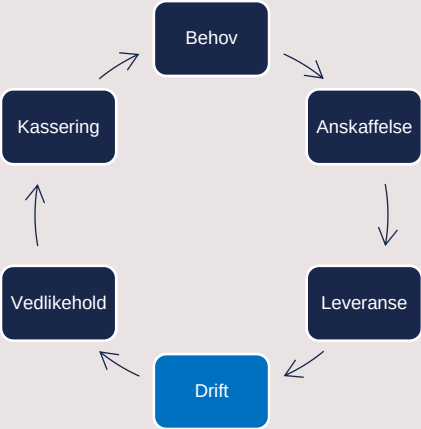

Devices

Condition	Name	State	Alert	Battery	Model	Department	Serial no	Next PM	Last connected	>
	Autoklave 1		Pumpefeil matevann	N/A	GSS67H		BAA110960			
	Autoklave 2			N/A	GSS67H		BAA110959			
	Autoklave 3			N/A	GSS67H		BAA110961			
	Kabinettvasker			N/A	9120E-001		WAA600044			

GETINGE

Recent Messages

Status	Date	Time	Program	Phase	Message
Started	5/12/2025	12:16 PM	Lekk.grad test	Ventemodus	Pumpefeil matevann

Equipment Message Details

Status	Date	Time	Program	Phase	Subphase	Message
Started	5/12/2025	12:16 PM	Lekk.grad test	Ventemodus	1	Pumpefeil matevann

Details of Message:

No return signal from motor controller when pump is activated. Not possible to fill water, evacuate chamber, or retract door seal

Possible Solutions:

- Check the motor protection.
- Check the contactors for the feed water pump.

PROSESSRAPPORT

MASKININFORMASJON

Maskin plassering BAA110960
Maskinnavn

PROSESSOPPSUMMERING

Programnummer P5
Programnavn Lekk.grad test
Dato 12/05/2025
Startet av hft
Avsluttet av 04700075
Prosesstart 12:41:51
Proseslutt 13:03:58
Syklusteller 403

PROSESSTATUS

IKKE GODKJENT

Dato Signatur

MASKININNSTILLINGER


PROSESSPARAMETERE
Programtype Lekk.grad test

LAST DATA
Ikke angitt


GETINGE

Vedlikehold

- Godt inntrykk

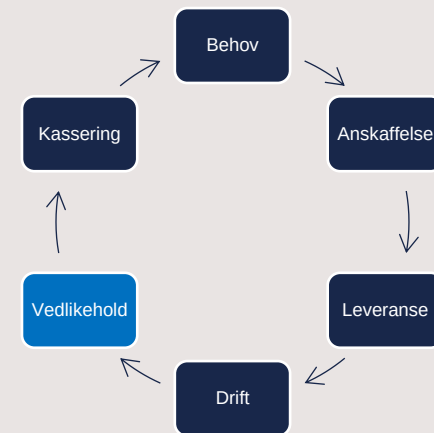

Kassering

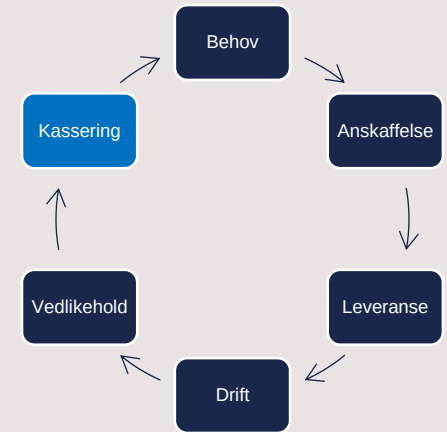

- Legges en plan i tide?
- Hvilke parametre styrer dette?



Oppsummert

- Indikatorer
- Digitale verktøy
- Opplæring



Takk for oppmerksomheten.

Kontaktinformasjon:

Are K. Blyverket

Mob: 98 28 11 74

E-post: are.blyverket@getinge.com

www.getinge.com

With a firm belief that every person and community should have access to the best possible care, Getinge provides hospitals and life science institutions with products and solutions aiming to improve clinical results and optimize workflows. The offering includes products and solutions for intensive care, cardiovascular procedures, operating rooms, sterile reprocessing and life science. Getinge employs over 10,000 people worldwide and the products are sold in more than 135 countries.



GETINGE

PASSION FOR LIFE